Please Read This Book First.

Item# 3001-2394-70
rev. 05/14

The logo for ORAQUICK features the word "ORAQUICK" in a bold, black, sans-serif font. The letter "Q" is stylized with a blue and green swirl graphic that overlaps it. The background of the entire page is a light blue and green gradient with abstract, flowing lines.

ORAQUICK®

HIV, Testing & **Me**

www.oraquick.com

If you have any questions about this HIV test or your test results,
call toll free 1-866-436-6527 or go to www.oraquick.com

**Please call our toll free number
(1-866-436-6527) regarding
any questions about:**

- Whether the test is right for you to take
- How to use the test
- How to read your test results
- What to do if you are not sure of your result
- What to do once you have your test results
- How to find local doctors or clinics for additional testing or treatment options

All calls are completely confidential.

**Or go to www.oraquick.com
for more information on:**

- HIV/AIDS
- Product Information
- Product use video
- Testing, counseling and care referrals
- Where to buy

If you have any questions about this HIV test or your test results,
call toll free 1-866-436-6527 or go to www.oraquick.com

Things to consider before taking the OraQuick® In-Home HIV Test

You probably have some concerns about your HIV risk and what your test results will be. This is normal, but by making the decision to learn about your HIV status you have taken a giant step towards **taking control of your future** and protecting your health. No matter what the result, knowing your HIV status is **important**.

You should know that it can take some time for the test to detect HIV in your body and that this time period can vary from person to person. This time period is commonly referred to as the "window period".

As with many tests, there is a possibility of a false negative reading. If this test is taken within the first **3 months** following a **risk event** or possible exposure to HIV and the result is negative you should take the test again **3 months** after the risk event. **See the list of risk events on page 3.** This will reduce the possibility of a false-negative result. **It is important to remember not to have any risk events during the time in between tests.**

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

What is a Risk Event?

A "risk event" is defined by any of the activities in the list below:

- Sex (vaginal, oral or anal) with multiple sex partners
- Sex with someone who is HIV positive or whose HIV status you don't know
- Sex between a man and another man
- Using illegal injected drugs or steroids
- Shared needles or syringes
- Exchanged sex for money
- Having been diagnosed or treated for hepatitis, tuberculosis or a sexually transmitted disease like syphilis

Knowledge is Important.

Let's start with some basic information about what HIV is and how it works in our bodies.

What is **HIV**?

In medical terms, **HIV** stands for **H**uman **I**mmunodeficiency **V**irus. This means that HIV is a virus that attacks the body's immune system, decreasing the body's ability to fight germs and ward off cancer. Germs can cause life threatening infections to a person whose immune system has been weakened by HIV.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

HIV is not **AIDS**. HIV is the virus that, if left untreated, can cause **AIDS** or "**A**quired **I**mmune **D**eficiency **S**yndrome". AIDS is a very serious condition in which the immune system has been badly damaged by chronic HIV infection and can lead to death.

There is currently no cure for HIV or AIDS, but medical treatment and a healthy lifestyle can **help you stay healthy** and improve your quality of life.

Who can get HIV?

Anyone can get HIV; men, women and children, people of any age, race or ethnic group, religion, economic background or sexual orientation. Today there are more than a million people in the U.S. with HIV and as many as 1 in 5 people don't even know it!

Can you help stop HIV?

Yes, you can **help stop** the spread of HIV.

HIV is transmitted from person to person during activities where contact with blood, semen or vaginal fluids can occur. By using a condom correctly every time you have sex, not using illegal drugs or sharing needles, and by encouraging your sexual partners and others to do the same, you can help stop the spread of HIV. Everyone should test at least once. Anyone who engages in high risk behaviors should test on a regular basis.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

How can you get HIV?

You can get HIV through contact with these body fluids:

- **Blood:** HIV is in blood, so anything you do that brings you in contact with another person's blood, like sharing needles or injection equipment, can transmit the virus.
- **Semen and vaginal fluid:** Because HIV is in semen and vaginal fluid, unless you use a condom correctly every time, you may be exposed to HIV during vaginal, oral and/or anal sex.
- **Breast milk:** HIV can also be passed from an infected mother to her child through her breast milk during breast feeding.
- Newborns of HIV positive mothers can be infected from exposure to blood at time of birth.

HIV is NOT spread by:

- Air or water
- Insects, including mosquitoes.
- Saliva, tears or sweat. There is no documented case of HIV being transmitted by spitting.
- Casual contact like shaking hands or sharing dishes.
- Closed-mouth or "social" kissing.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

Signs and Symptoms

Can you tell if someone has HIV? Can you tell if you have HIV?

No. **The only way to know for sure is to be tested for HIV.** People who have HIV can feel, look and act just as healthy as people who don't have HIV.

Remember! About 1 in 5 people who are HIV positive do not know that they are infected.

Protection & Prevention

HIV is completely **preventable**.

How can you **protect yourself** against HIV?

- Avoid having any form of sexual contact (vaginal, anal and oral sex).
- Do not use illegal drugs, especially if they need to be injected.

If this is not an option, there are good ways to **reduce the risks**:

- Always use a condom during vaginal or anal sex and barrier protection for oral sex.
- Have sex only with one partner that does not have HIV.
- Talk to your sexual partners about HIV and their status and testing history.
- Do not have sex with new partners until you both have been tested for HIV.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

- Do not have sex when you are using drugs or alcohol. Taking drugs and alcohol can cause you to do things that you wouldn't normally do.
- Do not share needles or injection equipment.

What if your Test Result is negative?

If it has been at least **3 months** since your last risk event, then you are likely HIV negative. As with any test, there is a chance of false results. For this test 8 out of 96 people who actually were infected with HIV reported a negative test result even though they had HIV. To reduce the chance of false results, be sure to use the test correctly. If you did not follow the directions carefully, you should take the test again. If your test result is negative and you engage in activities that put you at risk for HIV on a regular basis, you should test regularly. Knowing that you are HIV negative means that you can take steps to stay that way! It is important to learn about ways to reduce or **eliminate your risk of HIV infection** in the future.

The most important thing to keep in mind is that **HIV is preventable**. Protecting yourself against HIV is about knowledge. Understanding how you can get and avoid getting HIV is important to protect yourself and your partner(s).

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

What if your Test Result is positive?

It is very important to understand that if you get a positive result, a doctor, clinic or healthcare professional must confirm your OraQuick® In-Home HIV Test result.

There are also some things that you should know about HIV that may ease some of the stress or confusion that you may be feeling:

- You are not alone.
- Medical treatments are available to help people live long, healthy lives.
- Having HIV does not mean that you have or will get AIDS.

Learning that you may have HIV can be distressing. If you feel distressed remember you can get help by calling the OraQuick® Support Center toll free at 1-866-436-6527.

A positive test result is not the end of your life. While there is currently no cure for HIV, many people who are HIV-positive continue to live long and active lives. Ongoing research is finding better ways to treat HIV nearly every day.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

What should you do first?

It is important that you see your doctor or healthcare professional as soon as possible. You should let your doctor or healthcare professional know the result of your OraQuick® In-Home HIV test so they can **confirm your results.**

In addition to confirming your test result, doctors and clinics can provide counselors to talk with you about all of your concerns and feelings. If you need help finding a doctor or healthcare professional, please call the OraQuick® Support Center toll free at 1-866-436-6527. Your call will be completely confidential. They will ask for a zip code or city in order to locate doctors, clinics, health departments or other resources in your area.

Who will help you?

There are many people and organizations in communities around the country who are ready, willing, and able to help you. The key is to ask for help as soon as possible. If you would like to be linked to HIV-related medical or mental health services near you, please call the OraQuick® Support Center toll free at 1-866-436-6527.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

How can you stay healthy?

After an HIV diagnosis, medical care is necessary to stay healthy. Medical treatment is available to everyone, even those who cannot pay for it. Support groups are also available to help you find resources and programs in your area.

How to find out more about HIV or HIV prevention?

For more information, please call a Support Center representative at the OraQuick® Support Center toll free at 1-866-436-6527 or go to www.oraquick.com.

What is the Performance of this test?

In a clinical study, **4,999** people who were unaware of their HIV status were given the OraQuick® In-Home HIV Test to take home and use. The researchers compared the OraQuick® In-Home HIV Test results with laboratory test results performed by a trained professional. The laboratory based results show that a total of 96 people were HIV positive and 4,903 people were HIV negative. The comparison of results was as follows.

- 99.9% of people (4,902 out of 4,903) correctly reported a negative test result. The lab and the OraQuick® In-Home HIV Test found the same result. This means that 1 out of 4,903 people not infected with HIV reported a positive test result even though that person was really not infected with HIV. This is called a "false positive."

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

- 91.7 % of HIV positive people (88 out of 96) tested reported a correct test result. This means that 8 out of 96 people who actually were infected with HIV reported a negative test result even though they had HIV. This is called a "false negative."
- In addition only 1.1% of study subjects (56 out of 5,055) failed to obtain a test result.

As with any test, there is a chance of false results. Therefore, if your test result is positive, you must confirm your result with a doctor, clinic or healthcare professional. If your test result is negative and you engage in high risk activity on a regular basis, you should test regularly.

Knowing your status is important. Don't be afraid of your results.

By knowing that you are HIV positive you can seek treatment, stay healthy longer or improve your current health. It can also prevent you from accidentally passing it on to someone else. By knowing that you are HIV negative, you can help protect yourself and stay negative.

By educating yourself about HIV and AIDS and taking an HIV test, you are doing the right thing to protect your health and the ones you love.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

Please use the blank pages that follow to write down any questions that you may have for your doctor or healthcare professional.

Thank you for using OraQuick®!

www.oraquick.com

Manufactured by:

OraSure Technologies
Bethlehem, PA

If you have any questions about this HIV test or your test results,
call toll free 1-866-436-6527 or go to www.oraquick.com

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