
What your results mean to **You!**

What does a **positive** OraQuick® In-Home HIV Test result mean?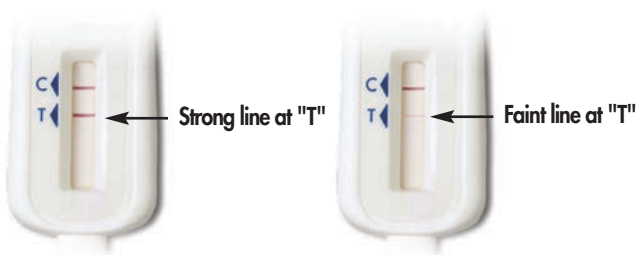

A **positive** test result has **TWO LINES**, (one next to the "C" **and** one next to the "T").

Any line, **even a faint line**, next to the "T" means you may have HIV.

If you get a positive result, you need to obtain a follow up test in a medical setting. A doctor, clinic or healthcare professional must confirm your OraQuick® In-Home HIV Test result.

www.oraquick.com

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

I may have HIV...

Although HIV is a serious infection, many people with HIV and AIDS live long healthy lives due to **new and effective treatments**. It can be distressing to get a **positive** HIV test result, but there are some things that you can do to **take control**. First you must **visit your doctor** if you have one. You may also **visit a local clinic or other healthcare professional to confirm your HIV test result**. Only then will you know if you have HIV.

Before you do anything else, please read this booklet and find out more about what your HIV test result means and what to do next.

What to do next about your positive test result?

- 1) It is important to contact your doctor or local health clinic and tell them that you have a HIV positive result using an OraQuick® In-Home HIV Test. You will need a confirmatory test.

If the confirmatory test is also positive, your doctor can help you get the treatment and care needed to stay healthy. Seeing a doctor is important even if you don't feel sick.

- 2) Until your HIV positive result is confirmed or proven false you must make sure that you don't do anything that may risk passing HIV to anyone else. See the **risk events** on page 6 that you should avoid.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

3) Should you tell someone about your HIV test result? This is a personal choice, but if you are not sure, your doctor, healthcare professional or health clinic counselor will be able to help you with that decision.

In addition to taking care of your health, you will need to find people who support you;

- a partner,
- friends,
- family members,
- clergy,
- etc.

HIV support groups are also available throughout your area and on-line. It is a good idea to discuss your support system with your HIV counselor or mental health provider before sharing your HIV status with others.

If you have any questions about finding a location for a HIV confirmatory test, or about your result, call toll free 1-866-436-6527 to speak to a Support Center representative.

What will your doctor do if you are HIV positive?

Your doctor or healthcare professional will run several tests to determine your current state of health and may want you to consider medication to help your body fight HIV.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

Your doctor or health care professional may also refer you to a doctor who specializes in HIV. You can get a recommendation from your regular doctor, local HIV clinic or hospital.

How will you pay for treatment?

The good news is that you may be able to get help paying for HIV treatment. The ability to receive payment for HIV treatment may vary depending on where you live. Talk to your doctor or call toll free 1-866-436-6527 for more information.

Remember that even if your OraQuick® In-Home HIV test result is positive, you will need to confirm that result through a doctor, clinic or healthcare professional. If your result is confirmed by a doctor, clinic or healthcare professional, it DOES NOT mean that you have AIDS or will ever get AIDS. AIDS is a very serious condition in which the immune system has been badly damaged by chronic HIV infection and can lead to death. Early detection and treatment can help prevent HIV from becoming AIDS.

New HIV drug treatments have made considerable advancements from the early years of this disease and can help you stay healthy for a very long time. Even if you cannot afford these medicines, these treatments are often made available to people who cannot afford them.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

How do I find out more about HIV?

For more information and to speak to a Support Center representative call toll free 1-866-436-6527.

What does a **negative** OraQuick® In-Home HIV Test result mean?

A **negative** test result has only **ONE LINE**. A line next to the "**C**" and **NO** line next to the "**T**" means that the OraQuick® In-Home HIV Test result is negative. If there is no line next to "**C**" your test is not working! You should contact the OraQuick® Support Center toll free at 1-866-436-6527.

As with many tests, there is a chance for false results. To reduce the chance of false results, be sure to use the test correctly. If you did not follow the directions carefully, you should take the test again to be sure your result is accurate.

If you have not had any risk events **within the past 3 months** and you followed the directions carefully, then you are most likely HIV negative.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

If you have had any risk events in the past 3 months you could be in what is called the "window period". The window period is when a person has been infected with HIV, but HIV tests cannot yet detect the infection.

If you think you may have been exposed to HIV through sexual contact or drug use within the past **3 months**, a follow up HIV test at a later date is always recommended. You should retest **3 months** following any "risk event". If you continue to engage in risk events that could put you at risk for HIV, you should test on a regular basis.

This product should not be used to make decisions on behavior that may put you at increased risk for HIV. See list of Risk Events below.

If you have any questions about your test result, or would like a referral to an HIV prevention counselor in your area, please call toll free 1-866-436-6527. A Support Center representative is standing by to help you.

Go to www.oraquick.com for more information on:

- HIV/AIDS
- Product Information
- Product use video
- Testing, counseling and care referrals
- Where to buy

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

What should I do next about my negative test result?

Tell someone or not.

This is a personal choice. Since your HIV test is negative, you can choose whether or not to share your result with sexual partners or others. But keep in mind that it is a good idea to help others and educate them about the importance of testing and knowing their HIV status.

What to do to stay HIV negative?

It is important to avoid HIV risk events as they increase the likelihood you may get HIV/AIDS.

What is a **Risk Event**?

A "risk event" is defined by any of the activities in the list below:

- Sex (vaginal, oral or anal) with multiple sex partners
- Sex with someone who is HIV positive or whose HIV status you don't know
- Sex between a man and another man
- Using illegal injected drugs or steroids
- Shared needles or syringes
- Exchanged sex for money
- Having been diagnosed or treated for hepatitis, tuberculosis or a sexually transmitted disease like syphilis

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

You tested HIV negative. Does this mean that your partner is HIV negative, too?

No. Even if you know your sexual or needle-sharing partners very well, you simply cannot be sure that they are HIV negative because of how they appear or what they may tell you. The only sure way to know is for them to take an HIV test and monitor their risk behaviors.

Many people think they are HIV negative because they don't want to believe that HIV could happen to them. About 1 out of 5 people who are HIV positive don't know they are HIV positive. The only way to know for sure is to take a HIV test.

Always talk to your sexual or drug using partners about their HIV status before you engage in any activity known to transmit HIV. Even if you and your partner have recently tested negative, it is a good idea to practice safe behaviors.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

What can you do to help stop the spread of HIV?

HIV is transmitted from person to person during activities where contact with blood, semen or vaginal fluids can occur. By using a condom correctly every time you have sex, not using illegal drugs or sharing needles, and by encouraging your sexual partners and others to do the same, you will help stop the spread of HIV. Everyone should test at least once. Anyone who engages in high risk behaviors should test on a regular basis.

You can get HIV through contact with these body fluids:

- **Blood:** HIV is in blood, so anything you do that brings you in contact with another person's blood, like sharing needles or injection equipment, can transmit the virus.
- **Semen and vaginal fluid:** Because HIV is in semen and vaginal fluid, unless you use a condom correctly every time you may be exposed to HIV during vaginal, oral and/or anal sex.
- **Breast milk:** HIV can also be passed from an infected mother to her child through her breast milk during breast feeding.
- Newborns of HIV positive mothers can be infected from exposure to blood at time of birth.

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

HIV is NOT spread by:

- Air or water
- Insects, including mosquitoes.
- Saliva, tears or sweat. There is no documented case of HIV being transmitted by spitting.
- Casual contact like shaking hands or sharing dishes.
- Closed-mouth or "social" kissing.

How do you find out more about HIV?

For more information call toll free to speak to a Support Center representative at 1-866-436-6527.


Go to www.oraquick.com for more information on:

- HIV/AIDS
- Product Information
- Product use video
- Testing, counseling and care referrals
- Where to buy

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

Take this page along to your doctor or health clinic

If you have a positive result when using the OraQuick® In-Home HIV Test, circle the picture that looked like your test result below:

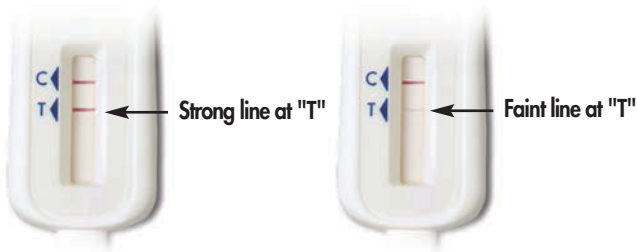Then pull out this page and bring it with you when you visit the doctor or local health clinic. Do not bring your used Test Stick with you. Dispose of it in the bag provided as instructed.

If your results do not look like the above examples or if you have any questions about this HIV test or your results, call toll free 1-866-436-6527.

www.oraquick.com

Manufactured by:

OraSure Technologies
Bethlehem, PA

If you have any questions about this HIV test or your test results, call toll free 1-866-436-6527 or go to www.oraquick.com

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